

Info balans U3 Light

U3-light har en fast bakaxel av kolfiber som inte kan flyttas

För att hitta en balansering som passar dig,

finns två olika modeller av denna stol, U3 light och U3 light L.

Mät avståndet A på din befintliga stol (bild 1), och titta i tabellen nedan vilken modell av U3 light, och vilken rygginfästning (bild 2) som din balansering motsvarar

För att göra en finjustering av balansen, har dessa modeller en justerbar rygginfästning där ryggen kan flyttas +/- 10mm. Det finns 3 positioner, för att fixera rygginfästningen.

Ryggen monteras i B som standard. Rygginfästningen kan, om så önskas, flyttas bakåt eller framåt 10mm för att finjustera balansen. Se bild 2.

Bild 1/Figure 1**Bild 2/Figure 2**

Balansering (A) "standard" U3

Modell

Position

Balancing (A) "ordinary" U3

Model

Position

50 mm	U3 light	A
60 mm	U3 light	B
70 mm	U3 light	C
75 mm	U3 light/L	A
85 mm	U3 light/L	B
95 mm	U3 light/L	C

ENG**Info balance U3 light**

The U3-light has a fixed rear axle made of carbon fibre.

We provide two different models of this chair to allow you to obtain the correct balance / center of gravity for your needs, the U3 light and U3 light L. Measure the distance A on your current chair (figure 1) and consult the table to see which model of the U3-light and which backrest attachment (figure 2) is equivalent to your current balance.

To adjust the balance of your chair, these models allow for adjustments of the backrest attachment whereby the backrest can be moved +/- 10mm. There are 3 positions on the tubing to allow you to fix the backrest position. The backrest is fixed into B position as standard. The backrest can be moved backwards or forwards by 10 mm to allow you to adjust the balance of your chair. See figure 2.