

Brake adjustment on Panthera X

For wheelchairs with a U2 light brake mounted.




Large adjustment

To make large adjustments of the brake, model U2 light, on Panthera X, first loosen the screws of the brake attachment (1) with a 15 mm spanner. Loosen both left and right side so much that it can be slid along the brake bracket (2).

Tighten the screws (1) light at the appropriate adjustment and try to brake. At the correct adjustment the brake will cut approx. 5 mm into the tyre, when the brake is activated.

When the correct position has been adjusted, tighten the screws (1).

NOTE Important! Do not tighten the screws (1) too hard because the brake attachment might break!

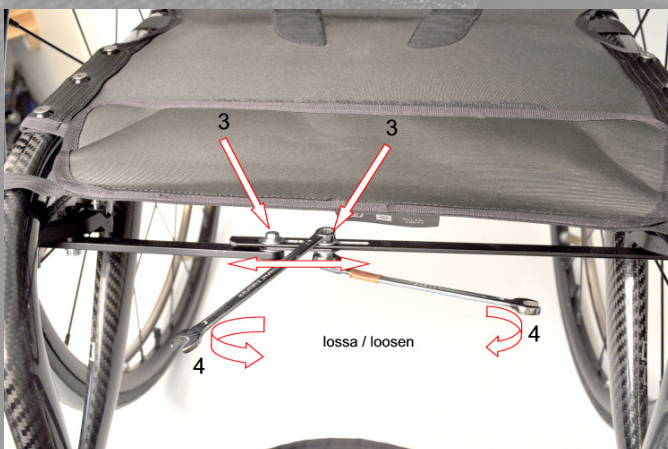

Small adjustment

If necessary you can make small adjustments to the brake by loosening the screws (3) with two 8 mm spanners (4).

Loosen the screws enough to adjust the bolts in slots. Adjust by changing the positions of the bolts in the slots and then tighten the screws. Activate the brake. At the correct adjustment the brake will cut approx. 5 mm into the tyre..



Correct adjustment

At the correct adjustment the brake will cut approx. 5 mm into the tyre, when the brake is activated. With fully inflated tyres (8 bar).