

panthera®

Info sheet 2013-04

03/06/2013


Mounting of push rim on our new 24" standard rear wheel



Our new 24" standard rear wheel makes mounting /removing push rims much easier since it is not necessary to remove the tyre and tube.

The push rim is held in place by 6 M5x12 mm Torx screws (size T25) that are tightened on the inside of the rim, seen from the side of the hub.

Older push rims that fit the older standard wheel will not fit the new since they have different inside diameters on the rim. However, push rims that fit the Spinergey wheels will fit the new standard rear

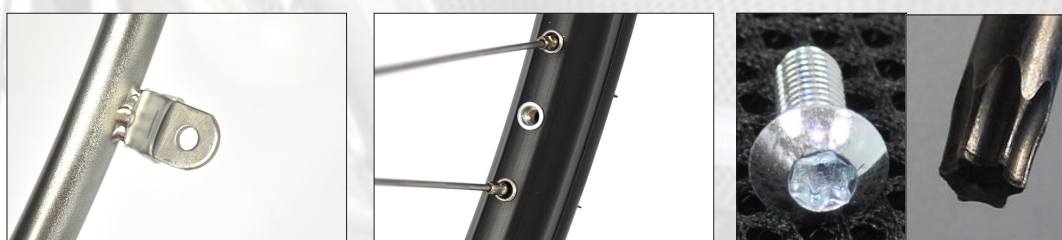

Six tabs on the push rim fit pre-drilled/threaded holes on the rim and are held in place by M5x12 mm screws that are tightened with a Torx tool (size T25).

Rear wheel	art.nr.	Push rim	art.nr.
Rear wheel 24" 6P without push rim	1300000	Push rim Titanium för 24"hjul 6P 510mm	1902411
Rear wheel 24" 6P without push rim och tire	1300000R	Push rim Titanium för 24"hjul 6P 510mm light	1902412
Rear wheel 24" 6P without push rim PU tire	1300001	Push rim ALU för 24"hjul 6P 510mm	1902430
Rear wheel 24" 6P Push rim Alu	1300010	Push rim MaxGrepp 24"hjul 6P 510mm	1903060
Rear wheel 24" 6P Push rim Alu PU tire	1300011	Push rim Geleplast för 24" 6P 510mm	1904204
Rear wheel 24" 6P Push rim Titanium	1300020	Push rim Geleplast för 24"hjul 6P	1904214
Rear wheel 24" 6P Push rim Titanium without tire	1300020R	Push rim PRC för 24"hjul 6P 510mm för 24" SPOX och 6P	1904304
Rear wheel 24" 6P Push rim Titanium PU tire	1300021	Push rim Para för 24"hjul 6P 510mm	1905124
Rear wheel 24" 6P Push rim MaxGr tunn	1300030	Push rim Tetra för 24"hjul 6P 510mm	1905224
Rear wheel 24" 6P Push rim MaxGr tunn without tire	1300030R		
Rear wheel 24" 6P Push rim MG PU tire	1300031		
Rear wheel 24" 6P Push rim GelePlast	1300040		
Rear wheel 24" 6P Push rim GelePlast without tire	1300040R		
Rear wheel 24" 6P Push rim Tetra PU tire	1300041		
Rear wheel 24" 6P Push rim PRC grå	1300050		
Rear wheel 24" 6P Push rim PRC grå without tire	1300050R		