



Transport protection for the Panthera X chassis

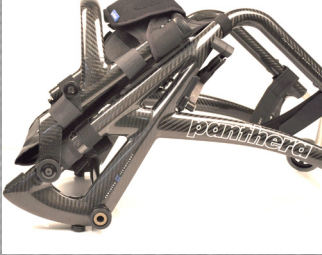

When lifting the wheelchair (e.g., in/out of a vehicle), there is a risk that the chassis may strike the asphalt and become scraped. To protect the most exposed parts of the Panthera X's carbon fibre chassis, a protective kit has been developed.
Weight when fully mounted on the chassis: **88 g**

NB! Fitting art.no. 9024007 entails chassis modification (i.e., drilling holes and fitting pop rivets). In order for the warranty to remain valid, fitting must be **done by Panthera**.

Components included in the protective kit:

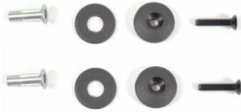



art.no. 9024007 Protective pads rear section X
NB! Fitting must be done by Panthera



art.no. 9024008 Protective rings rear axle X



art.no. 9024009 Protective sleeve X

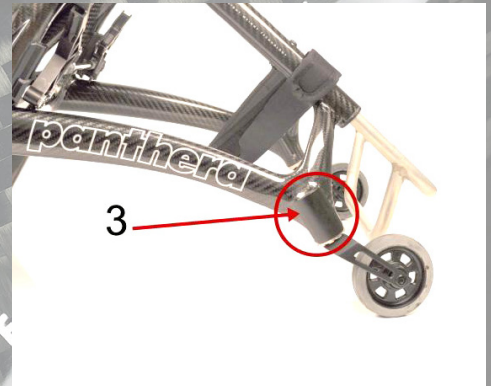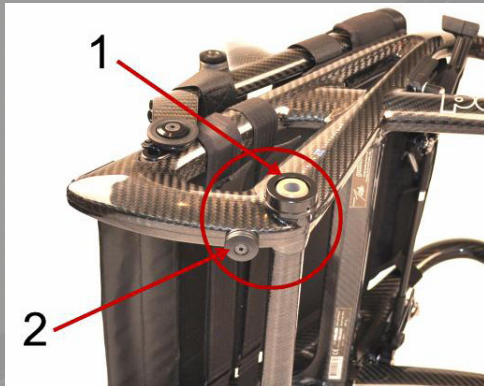

1) Where the rear axle protrudes, aluminium protective rings with screws should be fitted. **art.no. 9024008**

2) Plastic protective pads should be fitted with screws and pop rivets on the rear section of the chassis. **art.no. 9024007** (NB! This entails chassis modification that affects the warranty and must be done by Panthera.)

3) Aluminium protective sleeves should be glued to the outside of the front fork's mounting.
art.no. 9024009