

## UPGRADING OF STANDARD BAMBINO SEAT



UPGRADED BAMBINO SEAT WITH LONG BAR



OLD BAMBINO SEAT WITH SHORT BAR

All standard Bambino seat will be upgraded with a new long bar where the seat extender is attached, the same design as the abduction seat. This implicate an easier and more accurate adjustment of the seat depth without interference of the brake.


When you mount the new seat on an old standard chassis, without the pre-drilled holes for the new long bar, you need to use an awl to make screw holes in the chassis. (NOTE! Do not use a drill since it will damage the chassis)  
No change of art.no. and price.



ON OLDER CHASSIS THAT LACKS THE EXTRA HOLE FOR THE NEW BAR, YOU NEED TO USE AN AWL TO MAKE THE HOLE. WATCH VIDEO



THE FRONT SCREW OF THE BAR

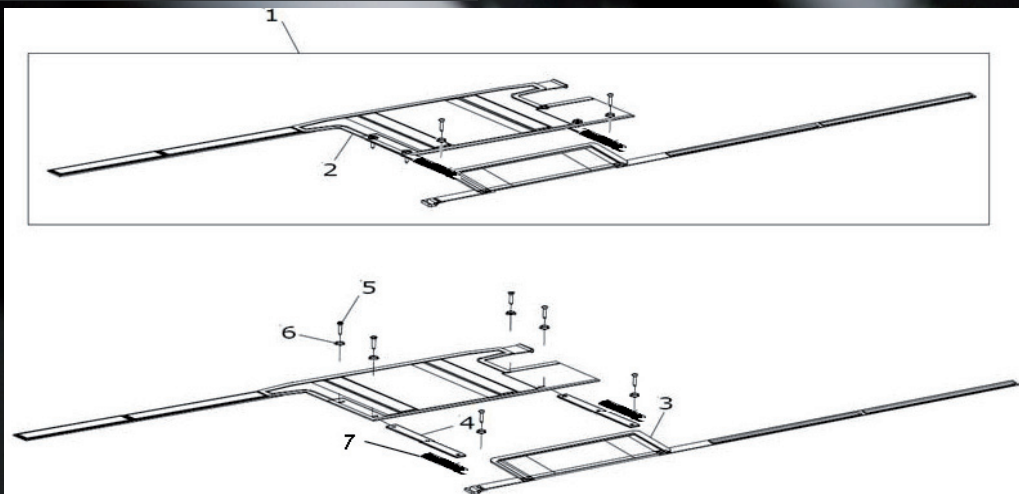

4. New long bar (art.no. 3890000)  
7. Cover bar (art.no. 3893502)