



Att: Prescribers
Technicians
Purchasers

Info sheet 2010-4

2010-10-14

ADAPTATION OF HEIGHT ADJUSTMENT OF THE FOOTREST (S2, U2, U2 light)

For some users the height adjustment of the footrest is not enough. Then you can cut off a distance of the frame tube and adapt the footrest with new holes. Then you can adjust the footrest to a higher position, approx. 20-30 mm.

NOTE! This adaptation is approved by Panthera AB and you may keep the CE label on the chair, if the work is done according to the instruction below!



Disassemble the footrest. Cut off the desired distance of the frametube, equal on both sides. Remove all burrs.



Make sure to leave at least 5 mm to ensure that the **welded seam is not damaged**.



Adapt the plastic plugs so they fits the shorter frame tubes.



Drill new holes (dia. 5,4 mm) on the footrest on both sides, so the it can be adjusted to the desired height, NOTE! The drillholes must not be closer to each other than C-C 8 mm.

Mount the footrest on the chair.