




FREEWHEEL- KIT FOR BAMBINO 3

Assembly instructions for:


Freewheel-kit Bambino 3
art.nr. 4234224, 4234227, 4234230, 4234233

Tools needed:
A long Allen key 4 mm


A reinforcement bar is mounted underneath the footplate.

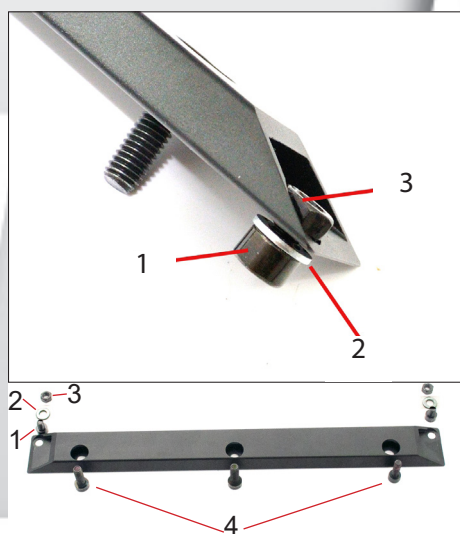

1) Fit screws (1) and washers (2) with nuts (3) at the ends of the reinforcement bar.

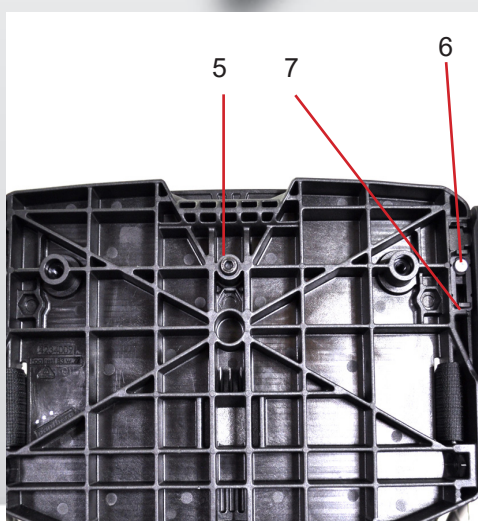

2) Remove screws (5) and screws (6), and locks (7)

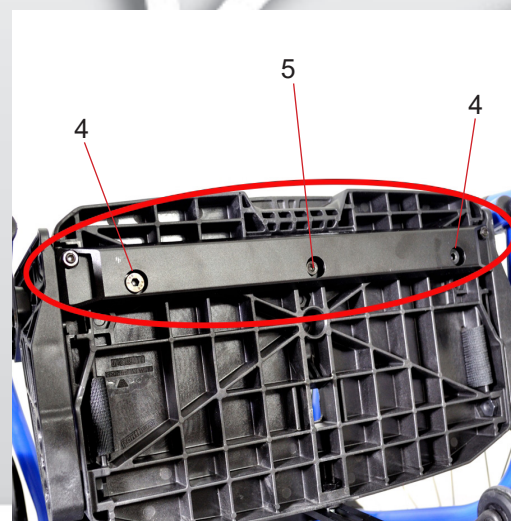

3) Mount the bar underneath the footplate with screw (5) in the middle and screws (4 Fig. 1) on each end.




4) Loosen the angle adjustment of the footplate.



5) Loosely mount the strut (8) with clamp (9) and screw (11) on the chassis.



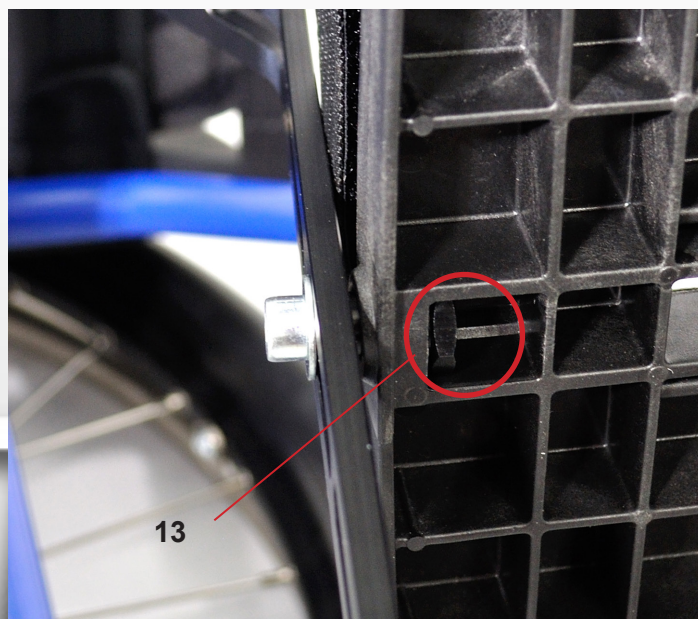


6) Fasten the strut (8) in the foot plate with screw (11), washer (12) and nut (13) which is inserted in the prepared slot in the rear of the foot plate. Adjust the angle of the footplate to the strut and tighten the screws on the clamp and footplate. Tighten the angle adjustment screws (Fig. 4) to 14Nm.



7) Cut a hole in the rubber mat for Freewheel.



For setting /adapting read the Freewheel Users manual.

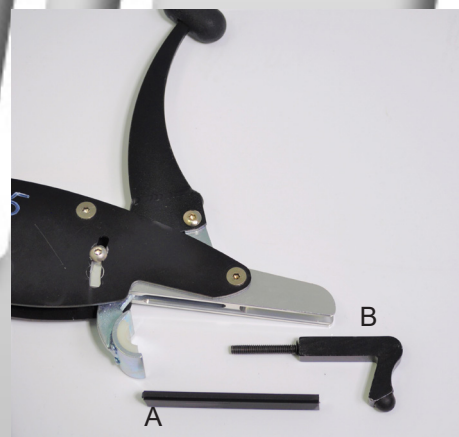

Mounting details for Freewheel on Bambino 3 footplate
A) Medium straight shim 5,25 mm thick, cut to 105mm
B) Short "rear end"