

TRAY-TABLE AND MAGNETIC HOLD BAMBINO 3



Tray table



Magnetic hold on rear wheel

Assembly instructions:
Tray table and magnetic hold Bambino 3

art.nr. 6810000 Tray table
art.nr. 6810008 Magnetic hold


Max. load 5kg

Tools required:
Allen key 3 mm, Pliers
Cutter / scissors


Placement of the brackets under the tray table for different seating widths of the wheelchair.
Letters "A" and "B" are stamped in the brackets. Table seen from below.




1) Width 24 cm. Use the **inner** holes for mounting.

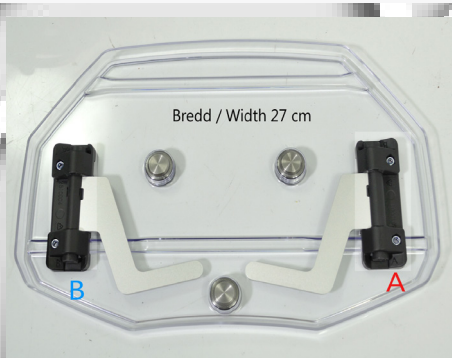


2) Width 27 cm. Use the **outer** holes for mounting.

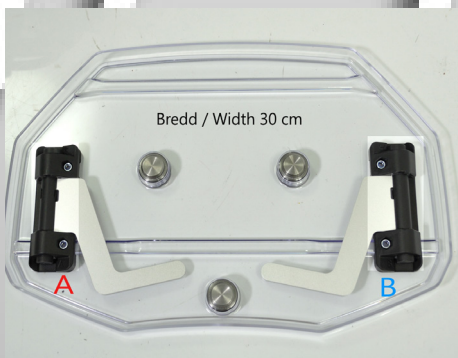

3) Width 30 cm. Use the **inner** holes for mounting. **Note** that the brackets have changed places between left and right compared to fig 1 and 2.

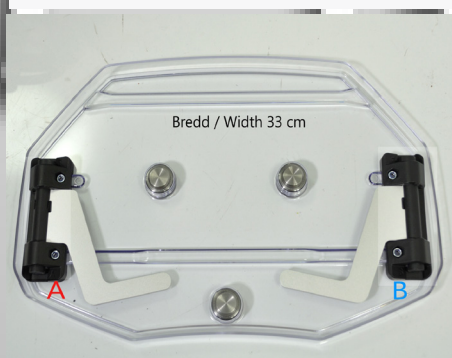

4) Width 33 cm. Use the **outer** holes for mounting. **Note** that the brackets have changed places between left and right compared to fig 1 and 2.

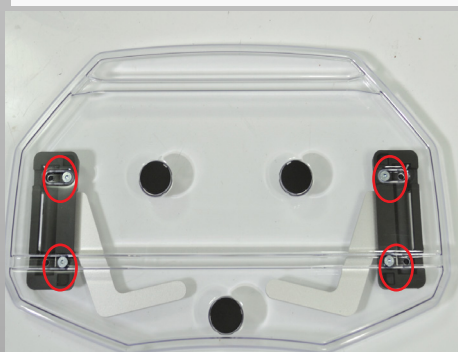

5) Turn the table over and screw in the screws. Make sure that the screws are tightened into the slot.



6) Place the table brackets in the holes in the side guards as shown in the picture and push down to the bottom.

Mounting of magnetic hold



7) Place the magnetic hold centered on the outside of the wheel. Rotate for each pair of holes are located over two spokes. If the wheels have spoke guards, first you need to drill 6 holes through the spoke guards (5 mm). Center the magnetic hold on the spoke guard and use as a template.



8) Thread the cable ties from behind to get the eyelet to end up on the back of the bracket. Tighten the tie holds hard. Cut the ends of the cable ties. If the wheel has spoke guards, the cable ties are pulled through both spoke guards and the magnetic bracket.



9) Attach the tray table to the wheel by aligning the magnets in the table and the hold on the wheel.