



BRAKE BAMBINO 3

Assembly instructions for:
Brake Bambino 3
art.nr. 4630201, 4630202, 4630301, 4630302, 4630300

Tools aquired:
Allen key 5 mm

NOTE! Before adjusting the brake, make sure that the tires are inflated inflated to **8 bar**.



1) Mount the brake on the chassis using a 5 mm Allen key and tighten the screw (1). Do not fully tighten the screw to a start.




2) Put the brake in the activated position by moving the brake lever (2) forward.



3) Slide the brake along the tube until the blade (3) touch the tire.



4) Check that the bracket (4) is perpendicular to the seat.

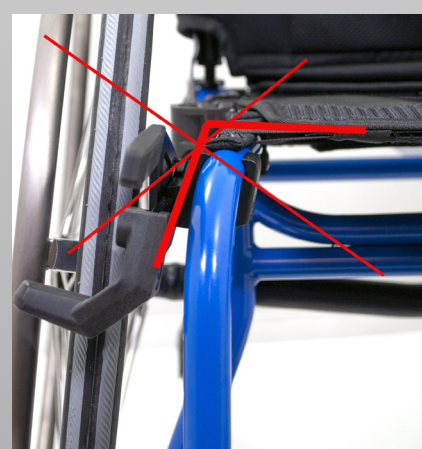



5) Release the brake by moving the brake lever (**Pos 2, Fig. 2**) upwards without changing the position of the bracket on the chassis.

Now slide the bracket (**4**) approx. 4 mm backwards on the chassis. Make sure that the clamp bottoms upwards towards the frame tube. Then tighten the screw (**1**) completely.

Also check that the angle between the bracket and the chassis does not change (**Fig 4**)

Check that the brake effect is correct.