

FOOT PLATE BAMBINO 3



Assembly instructions for:
Foot plate Bambino 3
art.nr. 42341WW

Tool aquired:
Allen key 4 mm
Allen key 5 mm

- The foot plate can be raised and lowered .
- The foot plate can be assembled in two different positions. A "low position" (default) and a "high position".
- The angle of the foot plate can be adjusted.
- Can be adjusted in two depth positions, front and rear.
- The foot plate can be folded backwards
- The foot plate can be locked in lowered position.



1) Height adjustment

On both sides, loosen (slightly) the screw (1) with a 5 mm Allen key. Adjust the footplate to the desired height, then tighten the screws.




2) Height adjustment -LOW position

Default is the footplate mounted in "low position". If necessary, you can turn the brackets and mount the foot plate in an extra high position (See pictures 3 and 4)



3) Height adjustment -HIGH position

The foot plate mounted in "high position" with brackets turned around.

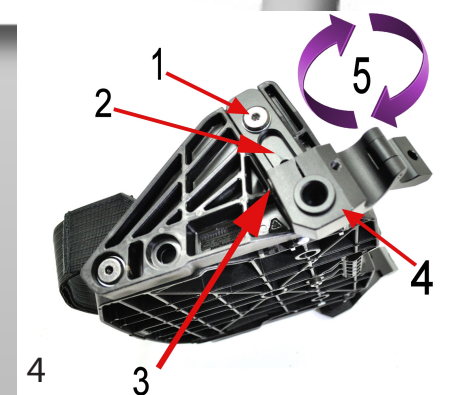


4) Switch between high and low position

1. Loosen the screw for angle adjustment (no. 1 in Fig. 5) with a 5 mm Allen key.
2. Loosen and remove screw and washer (1) with a 4 mm Allen key and nut completely on both sides.
3. Remove the footplate.
4. Insert nut on the inside of the footrest, turn the bar (2) 180 degrees, push the nut through the bar (2), fasten the screw from the outside.
- Repeat on the other side.
5. Tighten screws (1) from the outside.
6. Tighten the angle adjustment screws (1 in Fig. 5) to 15 Nm.



5) Angle adjustment


Loosen the screws (1) with a 5 mm Allen key and adjust the footplate to the desired angle. Tighten the screws to 15Nm

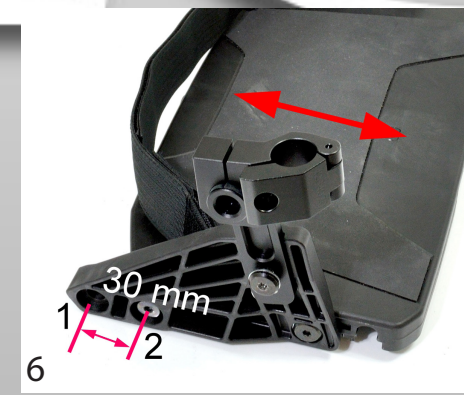

6) Depth adjustment

- Can be adjusted in two depth positions front (A) and rear (B)
1. Fold up the footplate (figure 7)
 2. On both sides, loosen and remove the screws (1) with 4 mm Allen key.
 2. Push out the nut sleeve from the outside on both sides and remove the footplate. Press the nut sleeve on one side and insert the foot plate in the desired position A or B, repeating on the other side.
 4. Reattach the screws (1) and tighten.
 5. Slide the catches (1 in Fig. 7) to the opposite side, lower the footplate.

panthera®

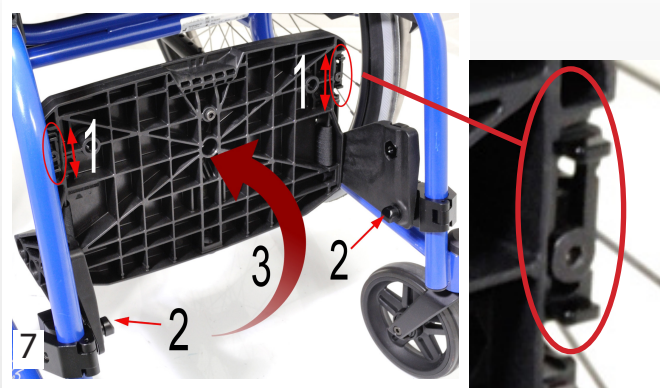

7) Fold the footplate

1. Loosen the catches (1) at the stop lug (2) by sliding them backwards, or forwards depending on the position.

2. Fold up the footplate until stop.

Down:

1. Fold down the foot plate until it stops and press until it snaps into the stop lugs (2).

2. Lock the footplate to the stop lugs (2) with the catches (1).