





BAMBINO HEADREST 3

Assembly instructions for:
Headrest Bambino 3
art.nr. 4401402, 4401403

Tools needed:
Allen key 4 mm
Allen key 2.5 mm

Mounting the headrest bracket

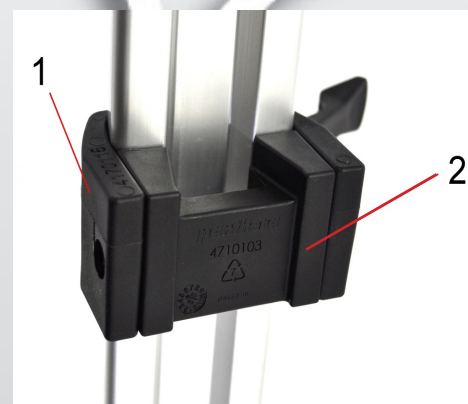
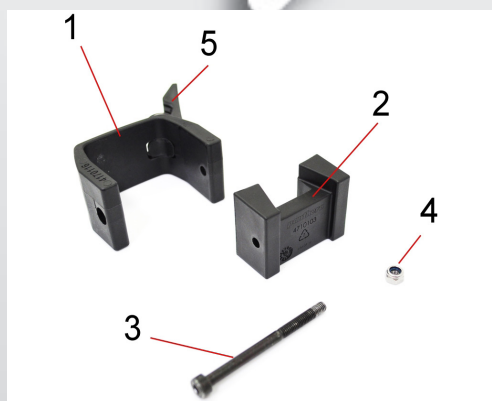


- 1)**
1. Clamp Bracket Headrest B3
2. Spacer piece Bracket Headrest B3
3. Bolt MC 6S M5x60
4. Nut M5
5. Lock lever Pushhandle S3

- 2)** Remove the clamp by removing screw no. 3 with a 4mm Allen
Remove Spacer piece no. 2
Make sure that lock lever no. 5 is open when making the assembly

- 3)** Thread the clamp (1) from behind over the aluminum profile of the pushbar.
Reassemble Spacer piece (2).
Thread screw (3) and tighten nut (4) firmly


When closing the lock lever (5), the clamp must be fixed on the aluminum profile and not be able to slide on the profile. When opening the lock lever (5), it must now be possible to slide the clamp up and down along the aluminum profile.

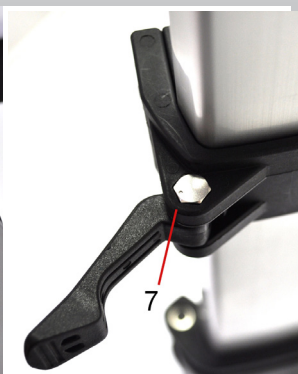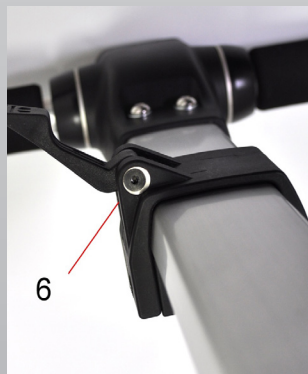

- 4)** If the clamp is not fastened sufficiently, the tension can be adjusted.


To do this, loosen the screw (6) with 2.5 mm Allen key, push out the eccentrics (7), turn both of these towards the center of the clamp and then retighten.

Mounting the headrest

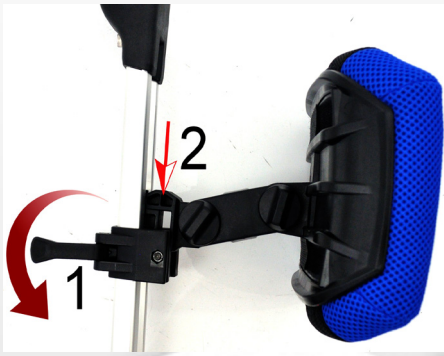

5) Open the bracket lock lever (1)
Thread the headrest mounting hook into the clamp.
Close locking lever (1)

Adjust headrest

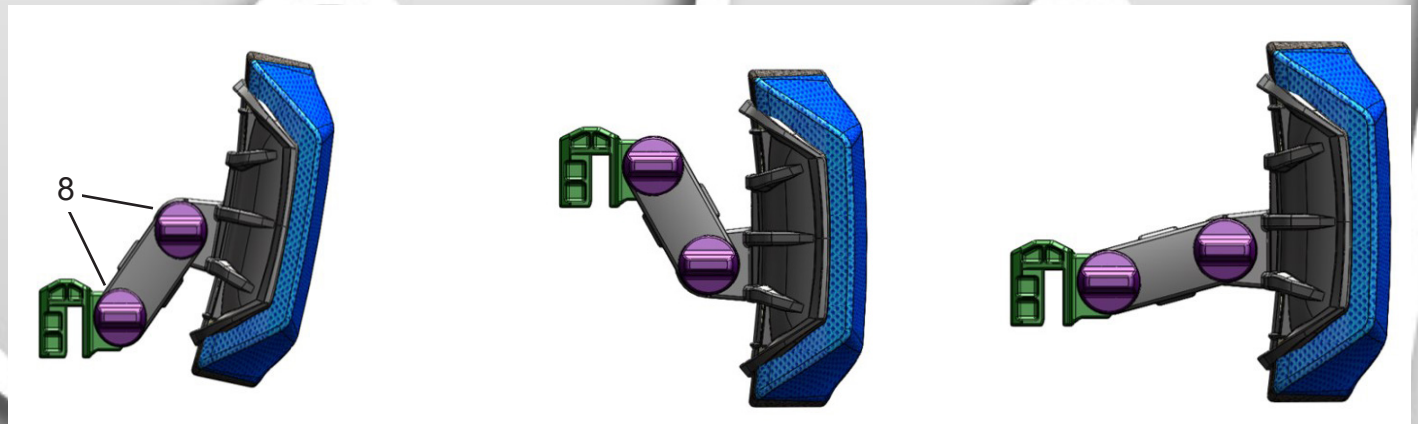

6) Depth, angle, and height
Loosen both knobs (8). Unscrew these about 3 turns. The stand can now be moved back and forth to desired position of the headrest cushion.



7) The headrest cushion can be mounted in high or low position. The difference will be 30mm (+ -15mm)
To shift the mounting of the head cushion, unscrew the knob (8) closest to the headrest cushion completely
Loosen the further knob a couple of turns.
Lever the stands apart - remove the headrest pad from the stands - turn it around and reattach.
Reattach and tighten both knobs.