


Assembly instruction Freewheel - how to turn the handle 180°




The handle on Freewheel needs , in some cases, to be turned around 180°, when it interfere with e.g. an adapted footrest to Panthera X.

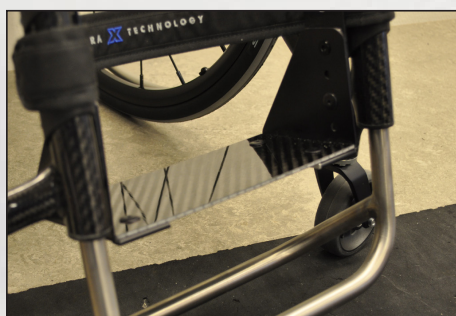


An example of an adapted footrest, where you need to turn the handle on the Freewheel around 180°.



Start by loosening this screw with Allen key 4 mm. Do not remove, just loosen..

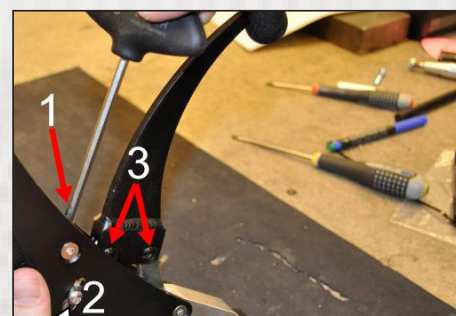

Loosen the screw (1) with Allen key 5 mm, to let the screw (2) slide in the slot and in that way expose the screws (3).

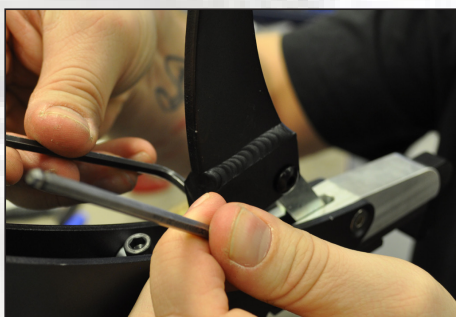


Remove the screws (3) using two Allen key 4 mm.



Put a drop of Loctite on the removed screws (3).



Grind a phase with a file on the edges without a phase, as similar as possible to those that are already there.

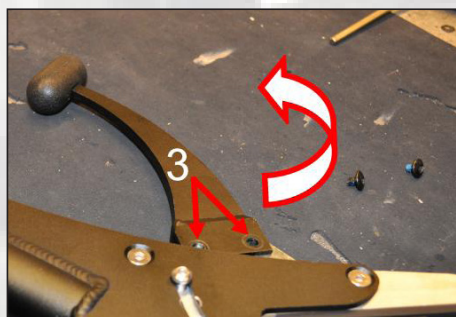

Turn the handle around 180° and reassemble the screws (3) with Loctite.



Then continue with the regular assembly instruction for the Freewheel to adapt it to the footrest.