0

No tools

TYRE SKINS reTyre TRACTION




Step 1




Center the mounting clip around a spoke and click it on the rim

Step2

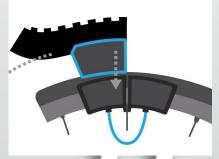

Attach one end of reTyre Traction skin to one half of the mounting clip and roll on to reTyre Traction. Make sure the reTyre Traction embraces the Tyre. It does not matter if you fasten front or back edge first and roll on, you choose what feels easiest. If you have side guards with fenders on the chair, it may be easier to put on the reTyre Traction from the front and roll backwards.

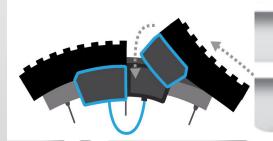

Mounting film

Step 3

After the full turn of the wheel, pull the end and attach the velcro ends to the other half of the mounting clip. Now the reTyre Traction is mounted and there should be a gap of 0-20mm in the seam.

Note!

To accomodate a variation in tire sizes, reTyre Traction has been design for a great ride with a gap up to 20mm.

