

panthera®

Assembly instruction

U2 light-brake on older X chassis

Valid for art.no. 3673300X
3683300X
3673600X
3683600X
3674200X
3684200X



The first versions of Panthera X had a U2 light onehand brake. The assembly of this brake differs from the Panthera X brake on later Panthera X chassis.

The wheelchair, **fig.1**, viewed from below with the rear wheels unassembled and the brake mounted.

The attachment for the brake **fig.2** on the chassis has 3 holes which enables to adjust the brake after the size of the rear wheel 24", 25" and 26". The hole closest to the rear is for 24".

Grab the brake **fig.3** and enter the screw "A", **fig. 8**, in the the brake, continue with washer "C" and "B", **fig. 4**, and after this, mount the spring house "D" where the spring should be hooked onto the screw on the brake. **fig. 5**. Enter the bush "E" on screw "A" acc. to **fig. 6**. Then enter screw "A" into the desired hole, **fig 2**. Tighten screw "A" with spanner 15 mm **fig. 7**. with adequate amount of force.

Repeat the procedure on the other side. Reassemble the rear wheels and test the brake.

