

## Assembly instruction amputation support

art.nr.6760501 / 2




### Assembly on S2


Unscrew the screws on the attachment.

Open up the attachment and put it around the frame tube on the correct side. This is a amp.support for the right side.


Tighten the screws slightly to be able to adjust later.

Place the support plate's tube into the attachment's tube.

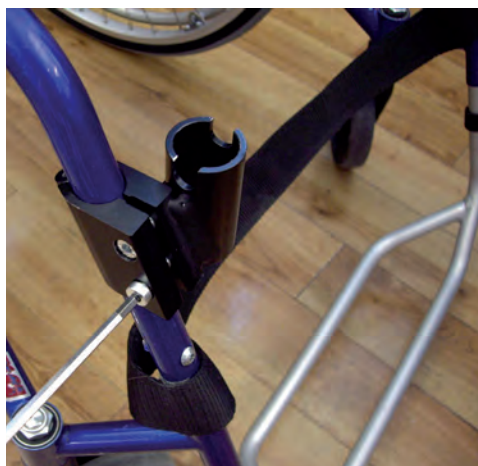

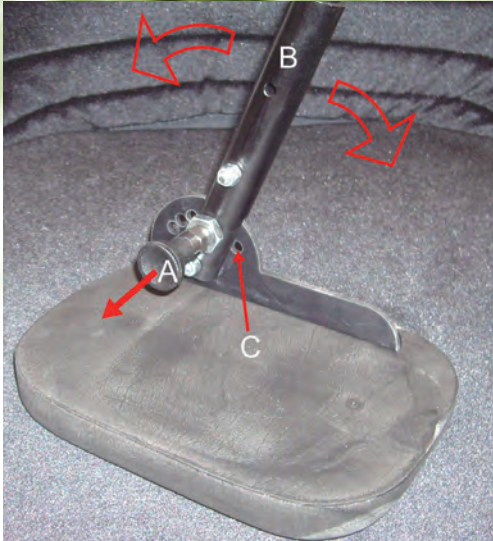

Adjust the angle of the plate by pulling the lever "A" straight out. Find the desired angle of the tube "B" and then push the lever back into one of the 7 holes "C".

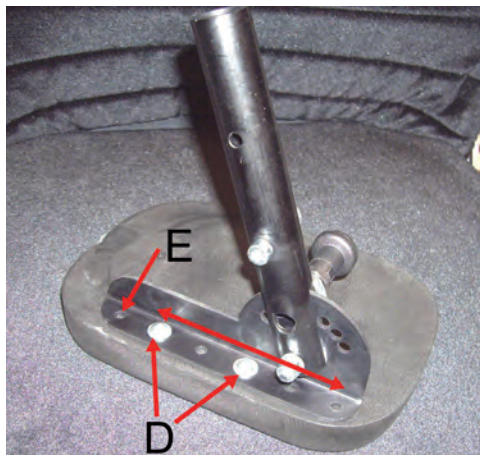

It's also possible to adjust the plate in depth. Loosen the screws "D" and find a new position. There are 6 predrilled holes "E" to choose from. Tighten the screws firmly.




When the correct setting is achieved finish the assembly by tightening the screws, pointed out, firmly.



## Assembly on S2 swing

On the S2 Swing the attachment is mounted on the swing-away footrest, see pic. It's not necessary to remove the footplate if there is a need to keep it.

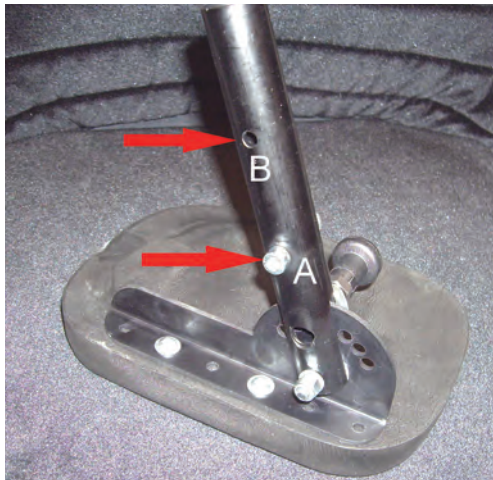

When using the amp.support on the S2 swing you need to move the stop screw "A" to position "B", to get the amp.support into the right level to the seat.



## Assembly on U2

The assembly on the U2 is made in the same way as on the S2 but you need to turn the plate around 180 degrees so it gets into the right position in front of the seat.