

TRUNK SUPPORT S3

Assembly instructions for:

Trunk support S3

art.nr. 6770200 Torso supports S3 complete pair

art.nr. 6770207 Assembly kit torso support S3 R

art.nr. 6770208 Assembly kit torso support S3 L

Tools needed:

Allen key 4 mm



QR link to
assembly video



1) Loosen the back rest upholstery to access easily to mount the trunk support.




2) Take apart the trunk support to make it possible to mount on the back rest..



3) Fold the clamping bracket around the back rest tube and enter the counter sunk screw.



4) Then fit the screw into one of the holes on the attachment.

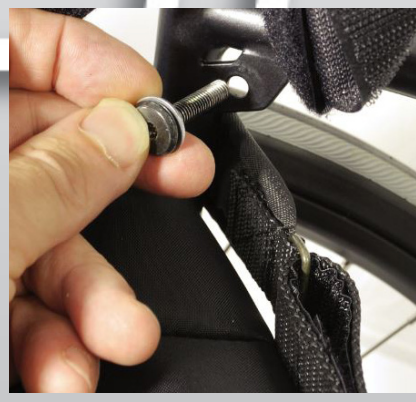

5) Mount the other screw into the hole on the ear that is welded onto the back rest tube.



6) Mount the bushing onto the screw..



7) Then fit the screw into the other hole on the attachment



8) Tighten the screw firmly with allen key 4 mm.



9) Mount the trunk support into the attachment.



10) Adjust the trunk support to desired height and then tighten the screws. Repeat the procedure on the other side if a left trunk support should be mounted. Finish by mounting the back rest upholstery.