

## TORSO SUPPORT S3 VER. 2

Assembly instructions for:

Torso support S3 ver. 2

art.nr. 6770300 Torso support complete pair VER2

art.nr. 6770307 Assembly kit torso support S3 R VER2

art.nr. 6770308 Assembly kit torso support S3 L VER2



QR link to  
assembly video

Tools needed:

Allen key 4 mm



1) Loosen the back rest upholstery to access easily to mount the torso support..



2) Disassemble the support for easy mounting on the backrest.



3) Enter the M6 screw into the hole in the reinforcement plate on the backrest.



4) Then fit the screw into the threaded hole in the attachment



5) Tighten the screw firmly with allen key 4 mm.



6) Place the bracket over the backrest tube.



7) Mount the cunter sunk screw into the bracket..



8) Tighten the screw firmly with allen key 4 mm.



9) Mount the torso support into the attachment, adjust the torso support to desired height and then tighten the knob. Repeat the procedure on the other side if a left trunk support should be mounted. Finish by mounting the back rest upholstery.