



BRAKE BLADE FOR S3 BRAKE

Assembly instruction for:
Brake blade for S3 brake


art.n0. 4630150, 4630146, 4630120, 4630123
4630124, 4630012

Tools needed:

Allen key 4 mm




[Link to video](#)

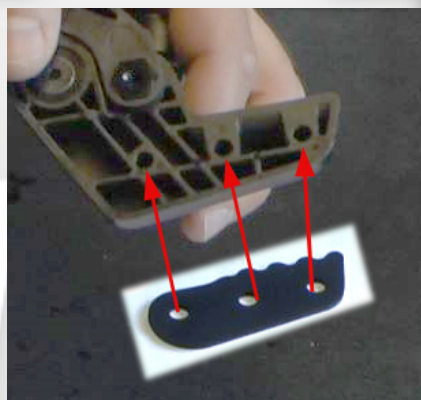

1) Place the holes of the brake blade over the holes on the underside of the S3 brake.



2) Mount the first screw and tighten it a few turns by hand.

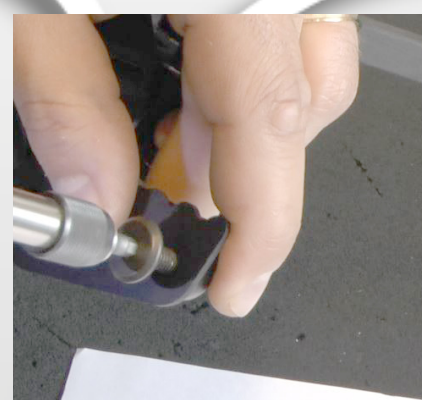

3) Tighten the screw almost to the bottom with a power tool or with a hand tool. Allen key 4 mm.



4) Continue with the rest of the screws..



5) Finally tighten the screws firmly to the bottom.



6) Finished. Repeat the procedure on the other brake.