

panthera®

Assembly instruction.

HOW TO ASSEMBLE YOUR WHEELCHAIR



Identify the parts included in your delivery

The parts and mounting instructions could vary depending on which chair and accessories you have ordered.
All parts and instructions are available at www.panthera.se



Chassis 1p




Rear wheels 2p



Castor wheels 2p



Back rest 1p

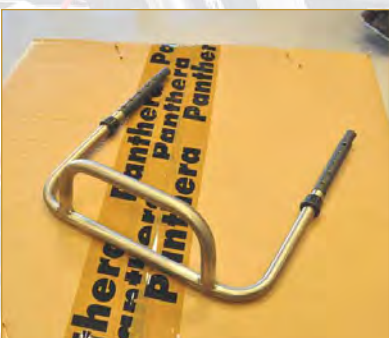

Foot rest 1p or footplates 2p



Brakes 2p or one hand brake 1p



Side guards 2p or without side-guards



Anti-tip 2p or without anti-tip



Push handles 2p or without push handles



Manual and tool kit 1p

1. Mount the rear wheels



Push the button in the centre of the hub to release the Quick release lock. Enter the wheel axle fully into the housing on the chassis. Release the button and check that the wheel is secured by pulling the wheel outwards. **Important !!!**

2. Mount the castor wheels




Enter the axle of the castor fork into the housing. Mount the washer on the axle and then the anchor nut. Tighten the anchor nut until there is no slack but the castor fork must still rotate easily.

3. Mount the foot rest




Apply some grease on and enter the footrest tubes into the chassis frame tubes. Adjust the height by choosing one of the holes (1) that should centre hole (2). Mount the screws and tighten them. Fit the plastic cover caps. Use a 19 mm spanner and tap it gently with a plastic hammer.

4. Mount the backrest

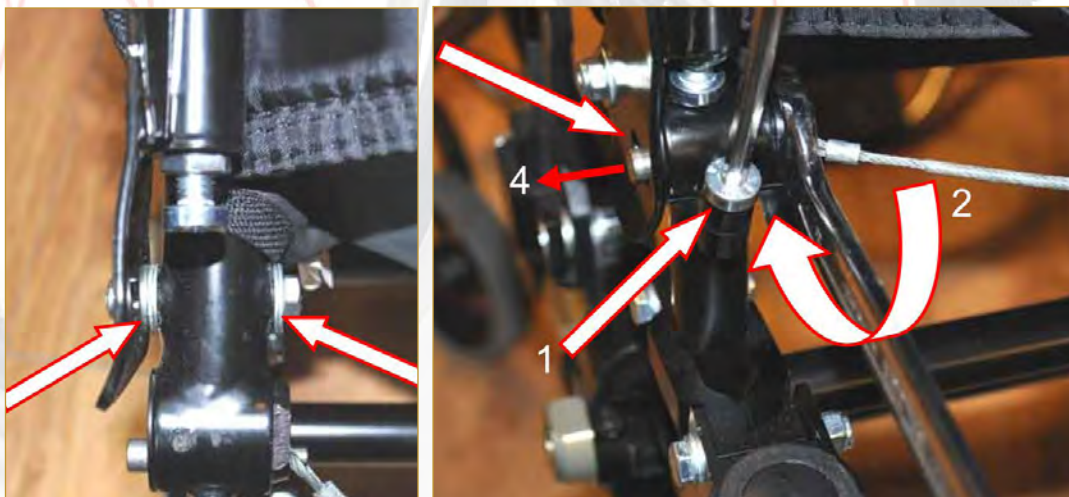

Mount the back rest on the chassis with bolt and washers as shown in the image. Loosen screw (1) and adjust the back rest lock (2) until the pin (4) pops out. Pull the wire several times to make sure the pin runs smoothly in the slot. Then tighten the screw (1). This should be done on both sides at the same time.

5. Mount the brakes

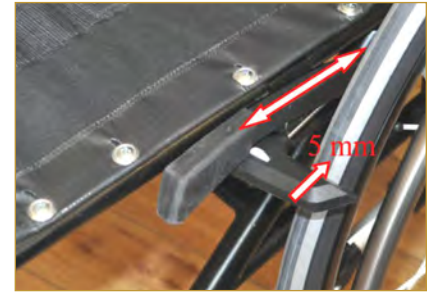

Place the brake bracket on the chassis tube. Tighten the screw slightly. Engage the brake and adjust it so it sinks in approx. 5 mm into the fully inflated (8 bar) tyre. Tighten the screw fully.

6. Mount the side guards



Place the side guards in the brackets beside the seat.

7. Mount the anti-tip device



Mount the anti-tip device as shown in the pictures. Use one washer on each side of the bolts.

8. Mount the push handles



Insert the push handle into the back rest frame tube. The threaded hole (1) should centre hole (2). Put the screw into the hole and tighten to secure the push handle.

This instruction is also available on www.panthera.se