

HOW TO REPLACE THE S3 CASTOR FORK



An instruction video can be found here:
<https://vimeo.com/141895412>



1) Remove the plastic cover from the locking nut. You might need a table knife to bend it loose.



2) Unscrew the nut with a 19 mm spanner.



3) Remove the old castor fork from the wheelchair.



4) Mount the new castor fork into the attachment on the wheelchair.



5) Mount the lower cover plate under the nut.



6) Put the cover plate and the nut onto the threaded castor fork axle.



7) Tighten the nut slightly with a 19 mm spanner.



8) Check that there is no play between the fork axle and the bearing housing and that it swivels easily.



9) Finally mount the plastic cover over the nut.