

PUSH HANDLE S3 HEIGHT ADJUSTABLE



Assembly instruction (adjustment page 2) for:
Push handle S3 Height adjustable
art.no. 4252060

Tools needed:
Allen key 4 mm
Allenkey 2.5 mm
Spanner 8 mm



1) Open up one of the attachments and place it around the backrest tube and fit the hole in the attachment over the knob on the tube. Note that there is a left and a right version of the attachment.



2) Stick the push handle tube into the attachment and lock it..



3) Squeeze the attachment to help fit the screw into the threaded hole. Tighten the screw a few turns by hand.



4) Finally tighten the screw firmly with Allen key 4 mm using a wrench



5) Done.

ADJUSTMENT OF LOCKING LEVER



1) If the locking of the push handle is insufficient, you can adjust this according to the following instructions..




2) Disassemble the push handle. Loosen this screw a few turns with an Allen key 2.5 mm.



3) Push the screw downwards until the nut on the underside gets visible



4) Adjust the locking force by turning the nut anti-clockwise (seen from the top) to increase it, and clockwise to decrease the locking force. Turn the nut a quarter of a turn, using an 8 mm spanner, tighten the screw and test it.

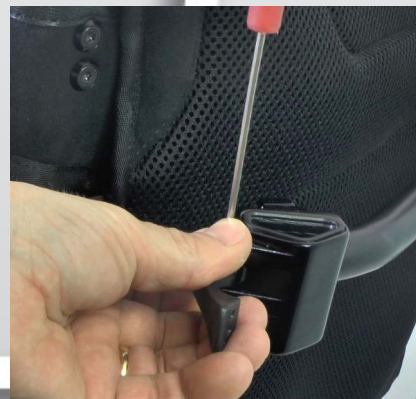

5) Tighten the screw and make sure that the nut slides into the attachment properly.



6) Mount the push handle and check the locking force. Repeat step 3-5, if adjustment is needed.