

PUSH HANDLE FOLDABLE S3



Assembly instruction for:
Push handle Foldable S3;
art.no. 4245000

Tools needed:
Power tool with bits with allen key 4 mm and a
screw driver.



1) Use a screw driver and push a hole in the plastic knob in the backrest tube, and pull it out.



2) Place one of the push handles into the backrest tube and pull away the upholstery to free the threaded hole.



3) Mount the screw into the hole and tighten it by hand for appr. 5 turns.



4) Finally tighten the screw firmly using the power tool with allen key 4 mm.

Repeat the procedure on the other side.



5) Done..