

panthera

Instructions for assembly

Seat bag

Page 1/1

How to assemble your seat bag:

1. When you receive your seat bag, the bag (see figure 1) will be enclosed along with a loose Velcro band.

2. Attach one Velcro strip on the seat bag to the Velcro on the upper side of the seat (underneath the cushion).

3a. If you have a standard chair or a U2 without single action brakes, you should attach the other Velcro strip on the seat bag to the Velcro on the underside of your seat. See figure 1.

3b. If you have a U2 with single action brakes, you should attach the loose Velcro band onto the chassis tubing as shown in figure 2, and then attach the other Velcro strip on the seat bag to this.



Figure 1



Figure 2



Figure 3

panthera

Panthera Production AB
Gunnebogatan 26, 163 53 Spånga, Sweden
Tel: +46 (8) 761 50 40 Fax: +46 (8) 621 08 10
panthera@panthera.se www.panthera.se