

## Seat bag U2-light

### Intended use

The idea is that the seat bag should remain attached to the cushion. When you lift the cushion you will then automatically have the bag with you and all your heavy phones, wallets etc. When you replace the cushion on your chair the bag will fall into place and fasten to the rear frame bar with the aid of magnets.



### How to use the seat bag.



The bag is attached under the cushion with Velcro band. When you lift the cushion up the bag comes with it.



The bag hangs down in front of the seat. Push the bag back, and the magnets in the bag fasten to the frame.



While you are in motion the bag is held firmly in place.

### How to attach the bag to the cushion

There are Velcro bands underneath the cushion. Use the self-adhesive band enclosed. Best results are achieved by sewing the band in place. The best position of the Velcro band is found by testing different positions to make sure the bag hangs down as far as you want it and that the end of the bag with the magnets reaches the rear frame bar of your chair.



Fasten the band at the front



..... or further back.



Then attach the bag to the Velcro band.