

Instructions for assembly Brake U2

1. The brakes are attached to the frame tubing directly underneath the seat upholstery.
2. First loosen the fastening screw (no.1) to enable you to separate the mounting clamps. This can be done using the enclosed T-key.
3. Then position the brake towards the front. Tighten the screws (no.1) on both sides so that the brakes are almost secure but can still be moved backwards or forwards by hand.
4. Pull to put the brakes on.
5. Move the brakes back so that the locking arms (no.2) are against the tyres.
6. Check that the brakes are positioned just as far forward on both sides. This can be done by measuring the section marked 'B'.
7. Push the brakes in to release and move them back around 5mm.
8. Before you tighten on one side, check that the brake is fastened level on both sides. You can check this by twisting the brake forward as far as it will go and then judging the central position.
9. You then tighten the screws securely.

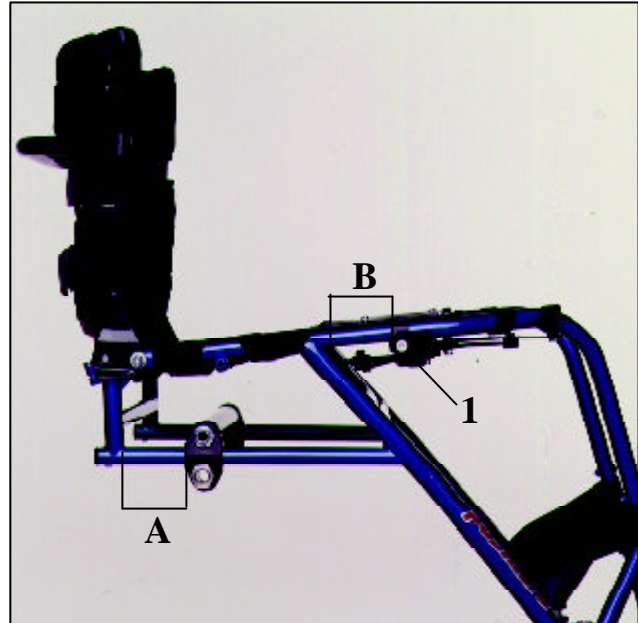


Figure 1

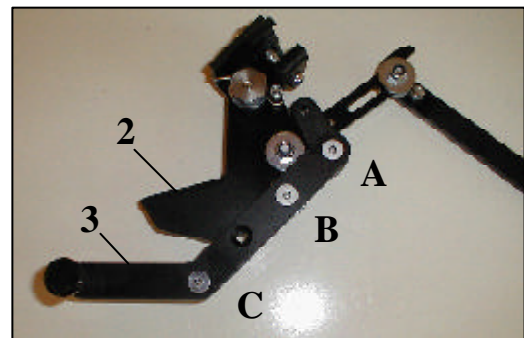


Figure 2

Important information

This brake does not fit in all positions. If the back axle is adjusted very far back or far forward the brake will not fit. You can check this by measuring the adjustment of the back axle. Measure the section marked 'A'. If this is less than around 20mm or more than 80mm the brakes will not fit these options.

Alternatives

Far back

If the back axle is adjusted in a position far to the back we have an alternative model that fits. We call this "the U2 rear positioned brake". The difference is that we exchange the locking arms (2) for an alternative design. Articles no. 3660121 locking arm, rear positioned U2 brake right, and 3660122 locking arm, rear positioned U2 brake left.

Far forward

If the back axle is positioned towards the front you can move the brake handle (3) back by assembling it in another way. See figure 2. Loosen the screws in positions A and B. Move the brake handle so that hole C moves to B and hole B to A. This brake handle was introduced in March 2000 as standard.