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CRUTCH HOLDER S3

Assembly instructions for:
Crutch holder S3
art.no. 9900301

NOTE! The mounting could vary depending on type of crutch.

Tools needed:
Small Philips head screw driver (Bits PH 0)
Torx TT10


Link to assembly
video

Link to user manual

1) Take the crutch apart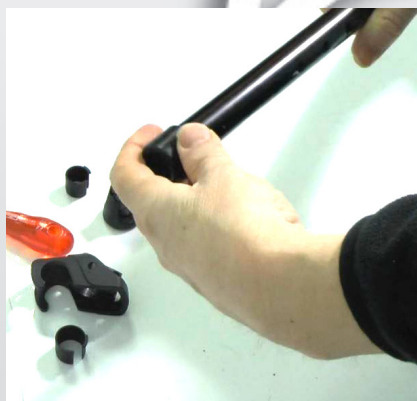

2) Remove the plastic hose from the upper part of the crutch.

3) Slide the upper attachment onto the upper part of the crutch. The upper attachment fits tube diameter 19 to 22 mm, including the plastic bushings.

4) Tighten one of the screws slightly. With a small philips head screw driver. (Bits PH 0) or Torx TT10


5) Remount the plastic hose.

6) Remove the rubber shoe from the lower part of the crutch.

7) Slide the lower, magnetic, attachment onto the lower part of the crutch. The lower attachment fits tube diameter 16,5 to 19,2 mm, including the plastic bushings. Don't tighten the screws yet.

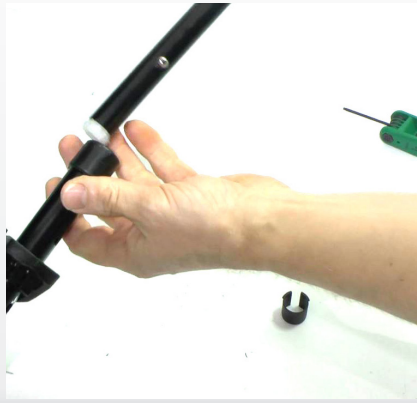8) Remount the crutch.

9) Hang the crutch with the upper attachment on the backrest tube. Turn the crutch until the handle of the crutch points backwards, away from the wheelchair.


10) Tighten the screws by turns with a small philips head screw driver (Bits PH 0). Use moderate force when tightening the screws. Don't tighten the clamps together completely. There should be a little gap between them.

11) Adjust the lower, magnetic, attachment in height until it is level with the chassis tube.

12) Tighten the screws by turns with a small philips screw driver. Use moderate force when tightening the screws.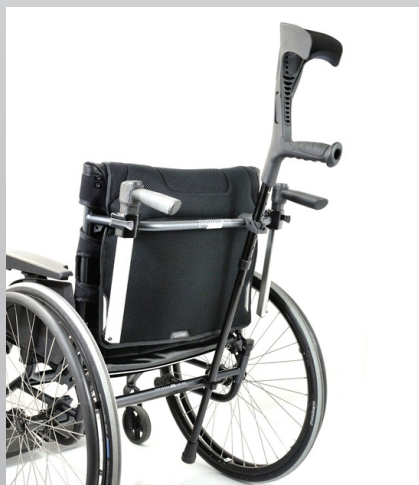

13) Attach the crutch to the wheelchair with its mounted attachments. Repeat the procedure on the other crutch, if there is one.