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Klart!

### Assembly instruction for: Raising-kit rear axle Carbon


art.no. 3302510 Assembly kit Carbon axle +10mm  
 art.no. 3302520 Assembly kit Carbon axle +20mm  
 art.no. 3302530 Assembly kit Carbon axle +30mm  
 art.no. 3302540 Assembly kit Carbon axle +40mm  
 art.no. 3302550 Assembly kit Carbon axle +50mm

QR Link to video

### Tools needed:

4 mm allen key, ruler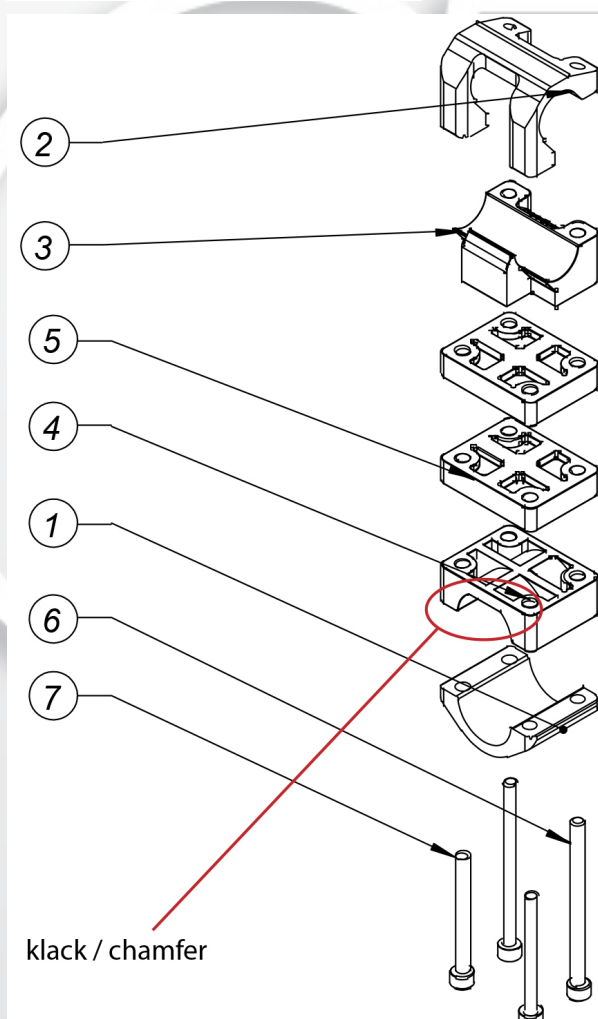

### Exploded view for raising-kit 30mm.

This assembly instruction shows a mounting of a 30 mm raise. With 2 pcs distances (5), One distance is 10 mm thick. Mounting of other heights is the same, just different numbers of distances, and different length of the screws (6, 7).

Rear axle **Carbon +1,5 cm** should be used together with the raising-kit.

Note that there is a small chamfer on part (4) that should face out from the chassis towards the rear wheels, on both sides.

The chamfer should line up against the chamfer on the rear axle to give the right rotation for correct camber angle.

CHAMFER ON REAR AXLE  
 AGAINST CHAMFER ON PART 4

When you raise the rear axle you must also compensate the height at the front of the wheelchair with other forks and castor wheels.


At Panthera's homepage you find an Excel-document that calculate the right combination of fork and castors.

[www.panthera.se/support/document/guide choice of castor fork.xls](http://www.panthera.se/support/document/guide%20choice%20of%20castor%20fork.xls)

1) Assemble the parts (5, 4 and 1) on screws (6, 7)

# panthera®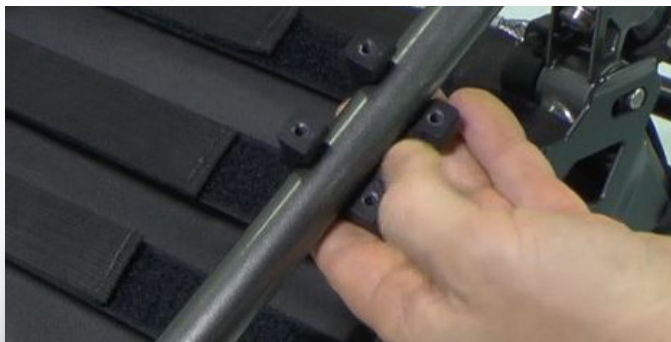

2) Place and hold the part of the attachment (2) around the chassis tube.

3) Put the other part (3) over the first.

4) Place the previously mounted parts on the attachment. The long screws should be on the outside and the chamfer on part 4 should face out from the chassis, towards the rear wheels.

5) Tighten the screws slightly with allen key 4 mm.

6) Enter the rear axle into the attachment. Make sure the chamfer at the end of the rear axle lines up with the chamfer on part 4.

7) Tighten the screws slightly just to keep the rear axle in position.

8) Continue with the other side and hold part 2 against the chassis tube.

9) Mount part 3 on top of the first part.


10) Add the distances (5) and part 4, make sure the chamfer is facing out from the chassis. Place the rear axle and line it up against the chamfer.

11) Place the attachment (1) and mount the screws through all parts. The long screws on the outside, against the rearwheels. Tighten all 4 screws slightly with allen key 4 mm.

12) Adjust the rear axle to desired setting "A" (COG). Use a ruler or similar. Importans to get the same measures on both sides. Standard is 60 mm on a S3 but vary depending on model.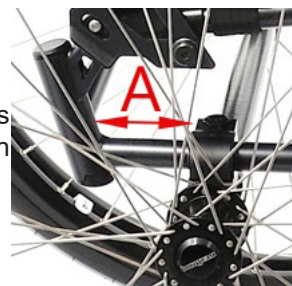

13) Tighten all 8 screws firmly with allen key 4 mm.

Done!