[Link to english](#)

KÖRBÅGE MICRO

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Allen key 5 mm

Link to video

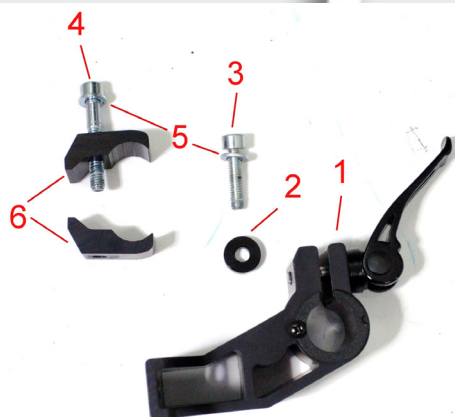
Fig. 1 Parts for the attachment for push bar Micro.

1) Attachment push handles H/A left	4250024
Attachment push handles H/A right	4600025
3) Bolt MC6S 88 M6x40 fzb	6002641
4) Bolt MC6S 88 M6x40 fzb	6002641
5) Washer BRB steel 6,4x12 fzb	8000612
6) Clamp standard	4600008

Fig. 2

At the rear there are two threaded hoses welded to the chassis that are for the attachment for the push bar. One for the left side and one for the right.

Fig. 3

Place the attachment, left (part 4, 5 and 6 in fig 1) around the tube, as in the image. Hold the part with the hand while attach the screw 1 into the attachment 2 with your fingers. Tighten until it stays in place.

Fig. 4

Use the parts shown in the image (part 1,2,3 and 5 in fig 1) and attach the screw 2 into the threaded hose 5 in the chassis.

Fig. 5
Tighten the front screw that holds the attachment around the tube with allen key 5 mm.

Fig. 6
Tighten the rear screw that holds the attachment to the th-readed hose with allen key 5 mm

Fig. 7
Place the push bar into the attachment and push the locking knob in order to get the push bar to enter the hose.

Fig. 8
Adjust it to desired height and lock it by pushing the lever down. If the lever is too loose or too tight you can adjust the pre-tension by turning it clock wise or anti clockwise. Turn it clock wise to increase the pre-tension.