BÅLSTÖD BAMBINO 3

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LATERAL SUPPORT BAMBINO 3

Assembly instructions for:
Art no: 4236000 Lateral support adjustable cpl

Tool needed:
Allen key 3mm
Allen key 4 mm
Torx T20
Thin flat head screwdriver.

Assembly of lateral support


1) Loosen the upholstery and fold it away

2) Take off the upper part of the upholstery which is mounted with 2 pieces of torx screws on each side.


3) Remove the plugs on top of the back tubes. Most easily performed with a thin screwdriver.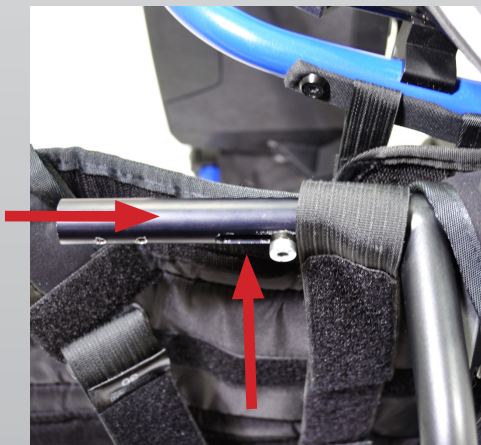


4) Place the wheelchair on its side with the back folded up, insert the "ratchet lateral support" (fig.1 pos 6) in the respective back tubes and fasten with screw and spacer (Fig 1 pos 7,8) with allen key 4 mm. The ratchet is also used to adjust the height of the lateral support.

Fig. 1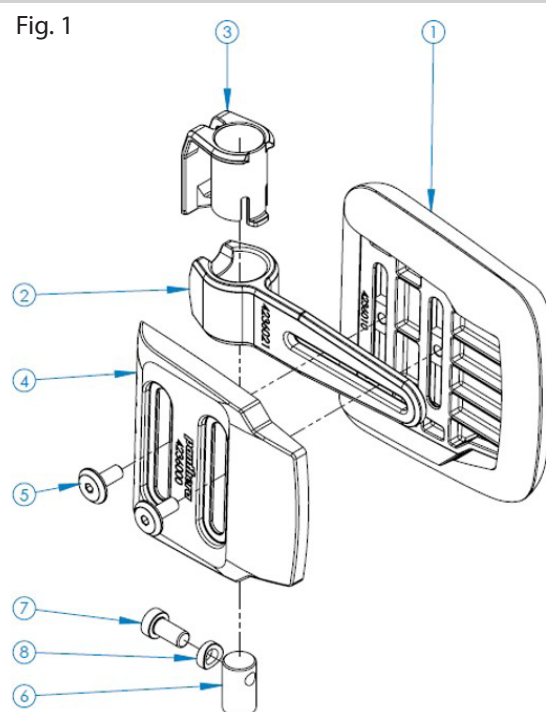


The lateral support can be mounted in two different ways to give different widths between the cushions.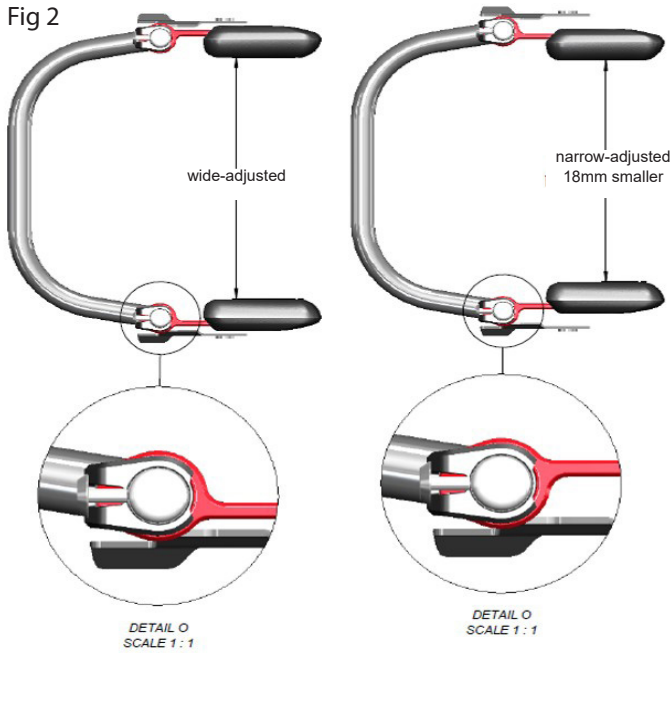

Decide width mounting

1. Select position on the metal arm Fig.2
2. Make sure the plastic bushing (fig1. Pos 3) inside the metal arm is entered from above (fig 3-5)
3. If the cushion is on the wrong side on the lateral support (fig.1 pos.1), fix this by removing the screws (Fig 1. Pos.5) that holds the cushion, use a 6 mm Allen key.
4. Remove the cover (Fig 1. Pos 4) and cushion and move the cushion to the opposite side, put the lid back and screw it on.

How to loosen plastic bushing

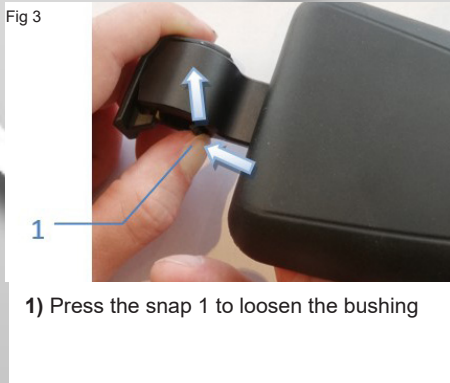- 1) Press the snap 1 to loosen the bushing

- 2) Pull snap 2, **max. 5 mm**, just enough to get the spike over the edge, to remove the bushing

- 3) Pull snap 2, **max. 5 mm**, Install the bushing from the other side

- 6) Mount the lateral support over the back tube until it stops against the bolt.

- 7) Pull out the snap, **max. 5 mm**, just enough to lift the spike over the screw.. Release the spike into the head of the screw.

- 8) The lateral support is in place. Refit the end plug and screw on the upper part of the upholstery, fold back the upholstery fabric.