KÖRBÅGE BAMBINO 3

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PUSHBAR BAMBINO 3

Assembly instructions for:
Pushbar Bambino 3
Art.nr. 4233000


Tools aquired:
Phillips screwdriver - small
Allen key 4 mm


1) Height adjustment of the upper clamp
If the back angle changes, the distance between the lower and upper clamps changes. To adjust this, you can change the position of the upper clamp as follows:
1. Loosen the screw marked on the illustration with a Phillips screwdriver until you can slide the upper clamp up or down.
2. Adjust the height of the upper clamp so that it fits on the backrest tube.
3. Tighten the screw again with 1 Nm torque.

2) Mounting of the push bar. Picture 2, 3, 4
1. Snap on the lower clamp (2) onto the rear chassis tube (3)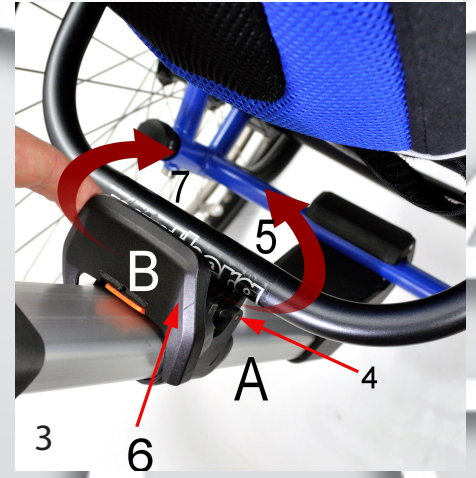


3) Mounting of the push bar cont. Picture 2, 3, 4
1. Open clamps A and B and place A under and around the back arch (5).
2. Bring clamp B (6) forward over the back frame and hook the front edge of clamp B over the front edge of clamp A.
3. Press down clamp B at the rear edge so that it locks around the back arch with a click

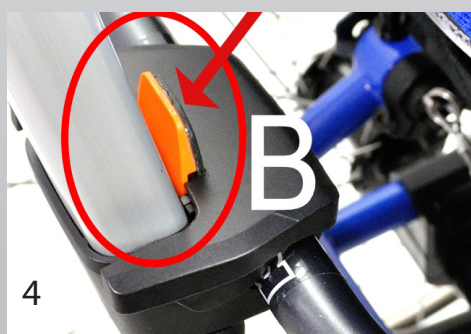
4) Mounting of push bar cont. Picture 2, 3, 4
Make sure that the orange latch locks securely over clamp B.
A click should be heard

5) Stabilize the lower push bar clamp Picture 5 and 6.
To stabilize the lower clamp against lateral movement, these two tube clamps are mounted on each side of the bracket on the rear chassis tube.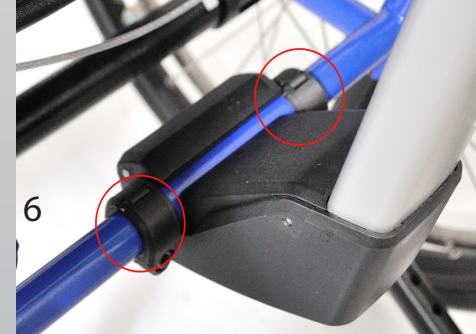


6) Stabilize the lower arch clamp cont. Figures 5 and 6
1. Mount the push bar
2. Attach the two tube clamps around the rear chassis tube on each side of the lower clamp. Use 4 mm Allen key.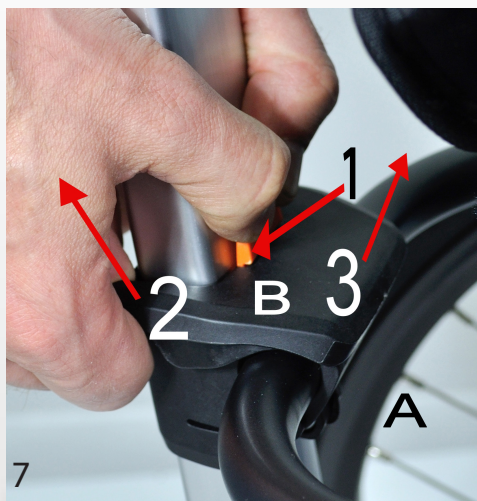

7) Disassembly of the pushbar Picture 7-8

1. Push the orange latch (1) against the post of the push bar.
2. Fold up (2) clamp B at the rear edge.
3. Loosen clamp B at the front edge from clamp A (3) which folds down so that it releases from the tube.

[Continue on picture 8](#)

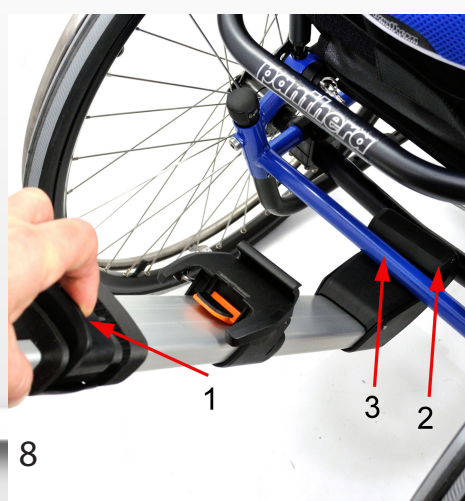8) Disassembly of the push bar. Picture 7-8

4. Squeeze the handle (1) to open the clamp (2) so that it can be detached from the tube (3)
5. Release the push bar from the wheelchair by pushing it down.

9) Settings of the pushbar

Height adjustment of the pushbar (step-less):

1. Fold out the locking lever (1) on the back of the pushbar.
2. Set the desired height and fold back the locking lever.

Angle adjustment of the handle (fixed positions):

1. Press the large button (1) at the top of the bow.
2. With the button (1) pressed, set the angle of the push handle, release the button. Pull the handle so that it clicks into a fixed position.